



ESENTA™ STING FREE ADHESIVE REMOVER SPRAY

EASY TO USE: FOLLOW THE BELOW STEPS

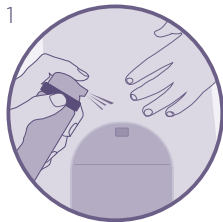


GETTING STARTED

Spray around the edge of the adhesive and wait a few seconds. You can spray at any angle.

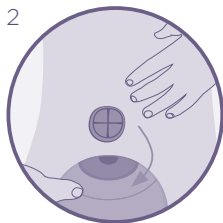

EASY REMOVAL

Gently peel adhesive away from the skin using additional ESENTA™ Adhesive Remover spray to ease removal, if necessary.

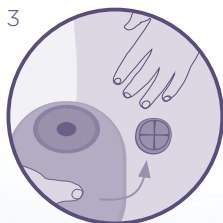

DRIES IN SECONDS

Once dried ESENTA™ Adhesive Remover will not affect adhesion of further dressings or appliances.



For more information please call
UK 0800 282 254
ROI 1800 721 721

See package insert for complete instructions for use.


©2022 ConvaTec Inc. ®/™ All trademarks are property of their respective owners.
AP-032787-MRL-GBL OST424v2



Printed on FSC mix recycled paper. FSC forests are managed with consideration for people, wildlife and the environment.


ESENTA™ STING FREE ADHESIVE REMOVER WIPES

EASY TO USE: FOLLOW THE BELOW STEPS

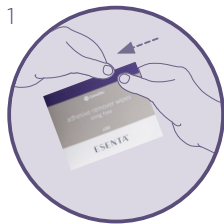


GETTING STARTED

Tear sachet to open and unfold wipe.

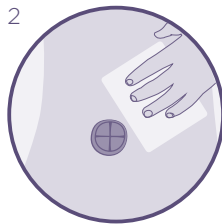

EASY REMOVAL

Rub the wipe over adhesive residues to remove.

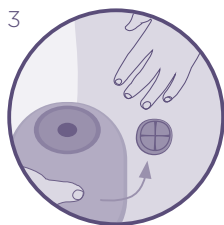

DRIES IN SECONDS

Once dried ESENTA™ Adhesive Remover will not affect adhesion of further dressings or appliances.



For more information please call
UK **0800 282 254**
ROI **1800 721 721**

See package insert for complete instructions for use.

©2022 ConvaTec Inc. ®/™ All trademarks are property of their respective owners.
AP-032787-MRL-GBL OST424v2



Printed on FSC mix recycled paper. FSC forests are managed with consideration for people, wildlife and the environment.