



Application Card



How to Use

Convatec Little Ones® One-Piece Stoma System

Baseplate/Pouch Preparation and Application



Measure the stoma size using the measuring guide provided. Trace the stoma outline you have selected onto the release paper on the baseplate. The small starter hole should be in the centre of your tracing.



Important: Before cutting, separate the front and back panels of the pouch to avoid cutting through the pouch. Pouch should contain a small amount of air. Cut along the tracing with curved, short-blade scissors.



Cut the centre hole the same size as the stoma, using guidelines on the release paper. Do not cut beyond the outer line.



Remove white release paper from the baseplate
If your child has a colostomy/ileostomy and you are using a drainable pouch, it is recommended that you close the pouch prior to application. Remove the release paper from the adhesive strip on the tail closure. Position the closure approximately $\frac{1}{4}$ " from the end of the pouch tail, on the side of the pouch without the baseplate. Press in place. Tightly fold the pouch tail upward four times. Tightly fold the tail closure ends toward the side of the pouch with the baseplate, ensuring the tail is smooth, flat, and not crimped.



If your child has a urostomy and you're using a urostomy pouch, make sure the tap is closed before applying the pouch. Close the drainage tube by bending the outlet spout over to meet the plug. Press the plug fully into the spout.



Place the pouching pouch gently in position over your child's stoma.



Press the baseplate against the skin, paying particular attention to the area closest to the stoma.



During application, maintain gentle pressure on the baseplate for 30 seconds to improve adhesion.

Pouch Removal



Once ready to remove the pouch, have a some water and some soft cloths ready to clean the skin. Hold the edge of the pouch with one hand and push down gently on the skin with your other hand to release the pouch from the skin and the baseplate, go slowly and gently, removing the baseplate and pouch together.

Tips:

- Remember to empty your child's pouch when it is one-third full.
- The use of soaps or cleansers in the pouch is not recommended. If desired, rinse only with tepid water.

For more information or if you have questions, call

0800 282 254

or visit our website at www.convatec.com/en-gb

