AutoSoft™ XC

Instructions for use

© 2023 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, AutoSoft and t:lock are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. AW-1001699_B

	Use-by (Expiry date) YYYY-MM-DD
	Do not re-use
	Sterilized using ethylene oxide
	Consult instructions for use
	Reference / model number
	Lot number
	Manufacturer
	Temperature limit for transportation: -20° C up to 55° C
	Unique Device Identifier
	Non-pyrogenic
	Open here
	Country of Manufacture: Mexico
	Date of Manufacture YYYY-MM-DD
	Global Trade Item Number
	Do not use if package is damaged and consult instructions for use
	Keep away from sunlight
	Keep Dry
	Medical Device
	Single sterile barrier system
	Prescription required, does not apply in all geographies
	Caution: see instructions for use
	Distributor

INDICATIONS FOR USE

The AutoSoft™ XC infusion set is indicated for the subcutaneous infusion of insulin administered by an external pump. The infusion set is indicated for single use.

INTENDED USE

The infusion set is indicated for subcutaneous infusion of insulin in the treatment of diabetes. The infusion set has been tested to be compatible with insulin approved for subcutaneous infusion. Refer to the insulin pump user guide for a list of compatible insulins and infusion sets.

PRODUCT DESCRIPTION

AutoSoft™ XC is an infusion device which combines an infusion set with an aid for insertion (inserter). As the infusion set is an integrated part of the inserter, AutoSoft™ XC is delivered assembled and ready for use as a single, sterile unit.

CONTRAINDICATIONS

AutoSoft™ XC is neither intended nor indicated for use in intravenous infusion of blood or blood products.

CLINICAL BENEFITS

- Continuous subcutaneous infusion of insulin in the treatment of diabetes.
- Less injections (insertions) compared to multiple daily injection therapy.

WARNINGS

- Carefully examine the instructions for use before inserting AutoSoft™ XC. Failure to follow instructions may result in pain or injury.
- AutoSoft™ XC infusion sets with t:lock™ tubing connectors should only be used with Tandem cartridges featuring the t:lock™ connector.
- AutoSoft™ XC is only sterile and nonpyrogenic if the insertion device is undamaged and unopened. Do not use if the insertion device is already opened or has been damaged. Ensure sterility by checking if the sterile paper and tamperproof seal are unbroken.
- When using AutoSoft™ XC for the first time, do so in the presence of a healthcare provider. Since different individuals have varying amounts of subcutaneous layer, ensure that an appropriate cannula length is chosen to fit your needs.
- Inaccurate medication delivery, site irritation or infection may result from improper insertion or maintenance of your infusion site.
- Do not re-insert the introducer needle into the infusion set. This could cause a tearing or damage of the soft cannula and may lead to unpredictable medication flow.
- Reuse of the infusion set may cause site irritation or infection and may lead to inaccurate medication delivery.
- Be sure that the needle guard is removed before insertion is made.
- If the soft cannula bends during insertion, apply a new AutoSoft™ XC immediately at a new site.

PRECAUTIONS

- Never try to fill or free a clogged tubing while the infusion set is inserted. This may result in unpredictable medication flow.
- When unwinding, release the tubing with caution as a hard pull of the tubing can result in damage to the infusion device. Ensure that the infusion set is properly in place in the inserter when the tubing is fully released (see figure 8).
- Do not leave air in the infusion set. Make sure to fill the tubing completely. Consult pump manufacturer's instructions for use.
- AutoSoft™ XC is a single-use device and should be disposed of immediately after use. Do not clean or re-sterilize.
- Do not in any way put disinfectants, perfumes, deodorants, cosmetics or other products containing disinfectants in contact with the connector or the tubing. This may impact the integrity of the infusion set.
- Replace the infusion set if the adhesive tape becomes loose or is displaced from its original site. Since the cannula is soft, it will not cause any pain if it becomes dislodged, and this may take place without notice. The soft cannula must always be completely inserted to receive the full amount of medication.
- The infusion set may be worn for a maximum of three days, or according to insulin labeling or healthcare provider's instructions, whichever duration is shorter.
- If your insertion site becomes irritated or infected, replace the infusion set and use new site until the first site has healed.
- Wash your hands before disconnecting and reconnecting the infusion set. Consult your healthcare provider on how to compensate for missed medication when disconnected.
- Never point a loaded insertion device towards any body part, where insertion is not desired.

RECOMMENDATIONS

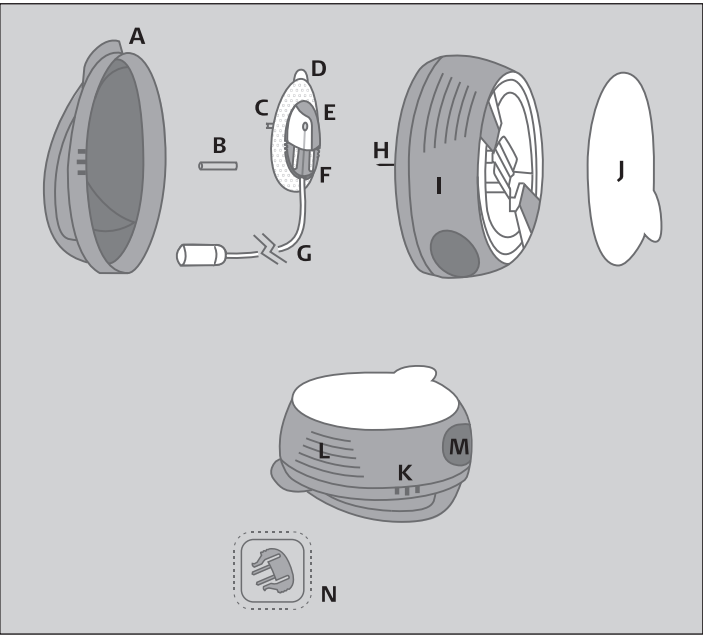
- Check your blood glucose level one to three hours after insertion of the AutoSoft™ XC. Measure your blood glucose on a regular basis. Discuss this with your healthcare provider.
- Do not change your infusion set just prior to bedtime, unless blood glucose can be checked one to two hours after insertion.
- If your blood glucose level becomes unexplainably high or an occlusion alarm occurs, check for clogs and leaks. If in doubt, change your infusion set since the soft cannula could be dislodged or partially clogged. Discuss plan to bring down your blood glucose level with your healthcare provider should any of these problems arise. Test your blood glucose level to make sure the problem is corrected.
- Carefully monitor your blood glucose level when disconnected and after reconnecting.
- If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

STORAGE & DISPOSAL

- Store infusion sets in a cool, dry location at room temperature (15-25 °C). Do not store infusion sets in direct sunlight, or in high humidity.
- Store and handle insulin according to the manufacturer's instructions.
- Put the lid back on the insertion device and dispose in a proper sharp container according to local laws, to avoid risk of needle stick.
- Dispose of a used infusion set according to local regulations for biohazardous waste.

CONTENTS

- A. Lid
- B. Needle guard
- C. Soft cannula
- D. Adhesive tape
- E. Cannula housing
- F. Connector needle
- G. Tubing
- H. Introducer needle
- I. Insertion device
- J. Sterile paper
- K. Three raised dots – press here when lifting the lid off (see figure 6)
- L. Lined indentations – press here when loading AutoSoft™ XC (see figure 10)
- M. Round indentations – press here simultaneously when inserting AutoSoft™ XC (see figure 12)
- N. Disconnect cover (see figure B)

PREPARATION AND INSERTION

Read all instructions carefully before using the AutoSoft™ XC infusion set. For information about pump therapy, possible errors, and potential risks and their causes, consult your pump manufacturer's instructions for use before connecting AutoSoft™ XC. When using AutoSoft™ XC, follow proper hygiene procedures. Please see the corresponding illustrations.

INSTRUCTIONS

1. Wash your hands before inserting AutoSoft™ XC.
2. Suggested insertion sites. Choose an insertion site as recommended by your healthcare provider. Do not use an area right next to your previous insertion site.
3. Clean the area of insertion with a disinfectant as directed by your healthcare provider. Be sure the area is air dry before inserting AutoSoft™ XC.
4. Pull the red tape to remove the seal.
5. Remove the sterile paper.
6. Press the three raised dots on each side of the lid firmly with one hand and lift up the lid with the other hand.
CAUTION: Be careful not to bend or touch the AutoSoft™ XC introducer needle when preparing the inserter and infusion set.
7. **Unwinding the tubing:** Carefully twist the beginning of the tubing out of the slot. Pull up gently. Now unwind the tubing from the insertion device by pulling it carefully upwards.
CAUTION: Do not touch the AutoSoft™ XC introducer needle when unwinding the tubing. Be careful not to pull too hard when the tubing is fully released. You could accidentally remove the infusion set from the inserter.
8. Connect AutoSoft™ XC to a cartridge and the pump. Prime infusion set. Ensure there are no air bubbles in the tubing. Consult pump manufacturer's instructions for use.
CAUTION: When priming the infusion set, hold AutoSoft™ XC with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.
9. Pull up gently to remove the adhesive backing paper.
10. **Prepare the inserter:** Place fingers on lined indentations. Press lined indentations on each side. Pull spring up until you hear a "click".
IMPORTANT NOTE: If the infusion set is not securely placed in the insertion device with the needle pointing straight ahead prior to insertion, pain or minor injury may occur. To avoid this, please ensure that the following

steps are taken gently so the infusion set is not loosened / removed by accident.

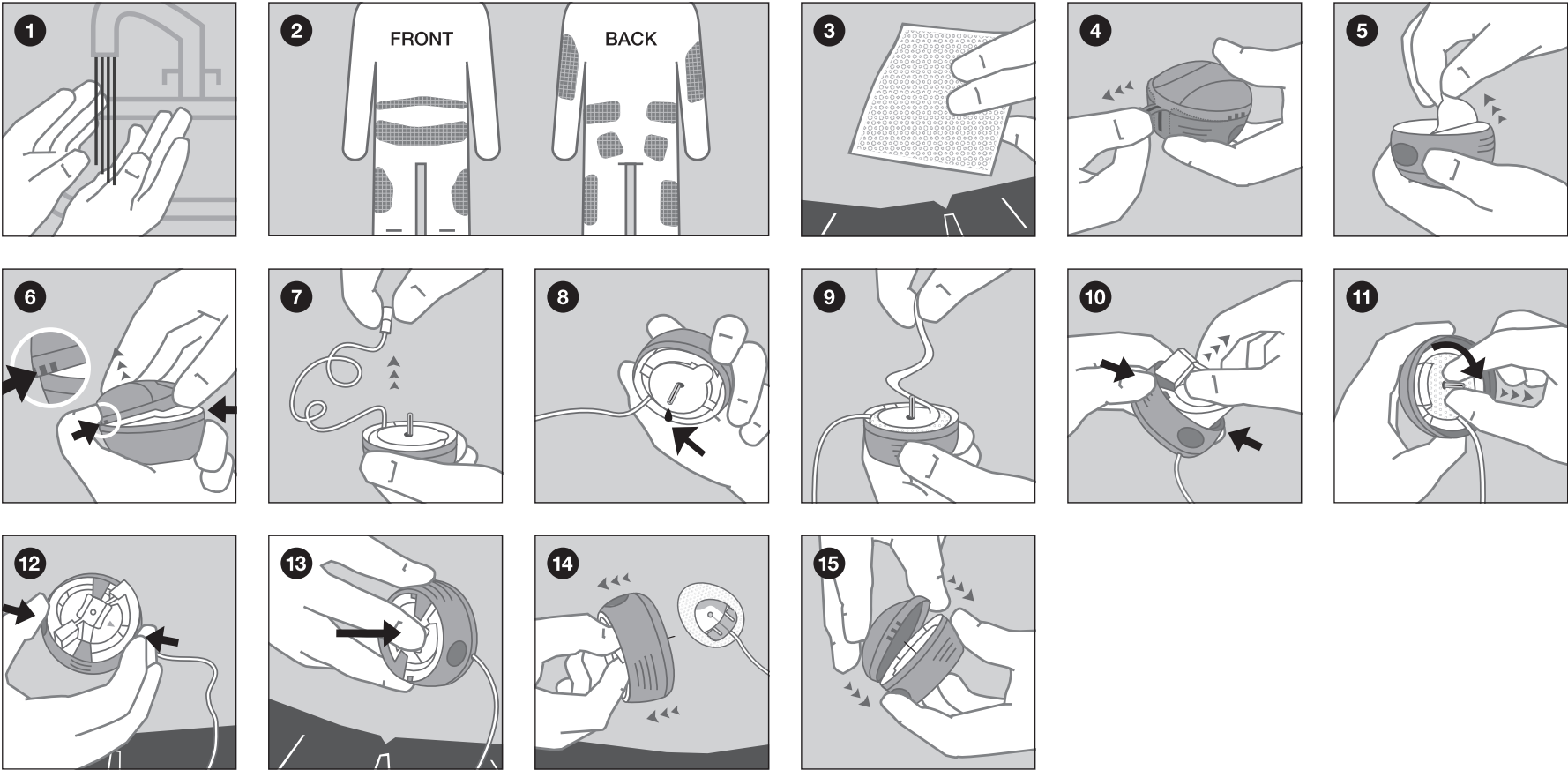
11. Carefully remove the needle guard by gently twisting and then pulling it. Check that the soft cannula does not exceed the introducer needle.

CAUTION: Before positioning AutoSoft™ XC over a clean insertion site, place the tubing in the slot to ensure that the tubing is not caught underneath the infusion set during insertion.

12. **Insert Autosoft™ XC:** Position AutoSoft™ XC over the insertion site. Simultaneously press the round indentations on both sides of the insertion device to insert AutoSoft™ XC.
13. Push gently on the center of the inserter to secure the adhesive tape on the skin.
14. Remove the inserter/introducer needle by grasping the center and pulling it gently back. Gently massage the tape onto the skin. Fill the empty space in the cannula with the following amount of insulin:


U-100 insulin:
6 mm: 0.3 units (0.003 ml)
9 mm: 0.5 units (0.005 ml)

15. Push the lid back in place until you hear a "click."


DISCONNECT

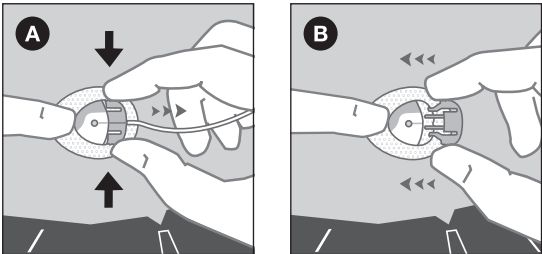- A. AutoSoft™ XC allows you to temporarily disconnect your pump without having to change the infusion set. Hold the adhesive in place by placing a finger just in front of the cannula housing, and gently squeeze the sides of connector. Pull the connector needle straight out from the cannula housing.
- B. Insert the disconnect cover into the cannula housing until you hear a "click".

RECONNECT

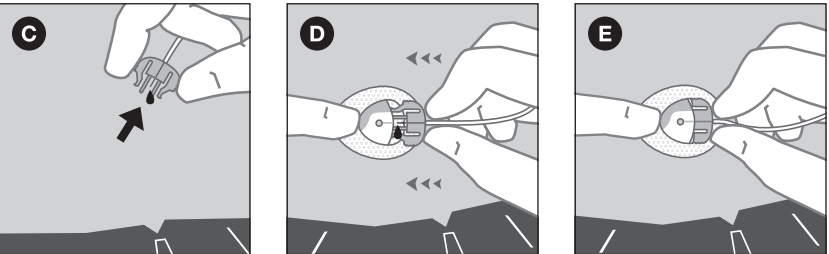- C. If necessary, prime the infusion set until medication dispenses from the needle.
CAUTION: When priming the infusion set, hold AutoSoft™ XC with the needle pointing down.
- D. Remove the disconnect cover from the cannula housing by pressing the grips. Place a finger on the adhesive tape just in front of the cannula housing.
- E. Push the connector needle straight in until you hear a "click".

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